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



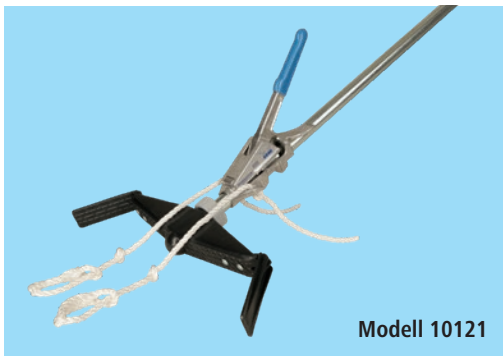
HK Calving Aid
Operating Instructions

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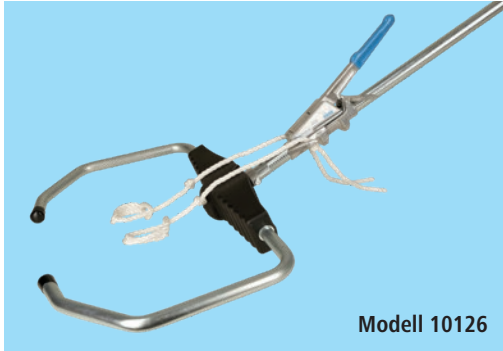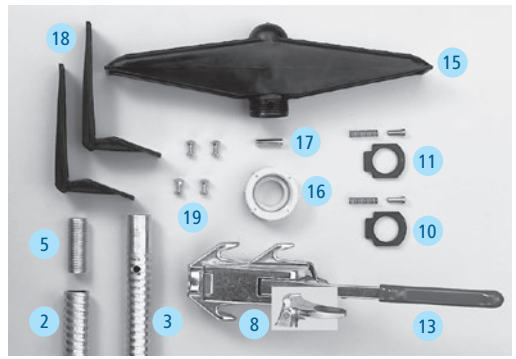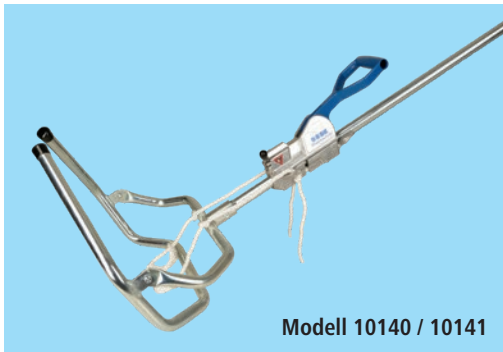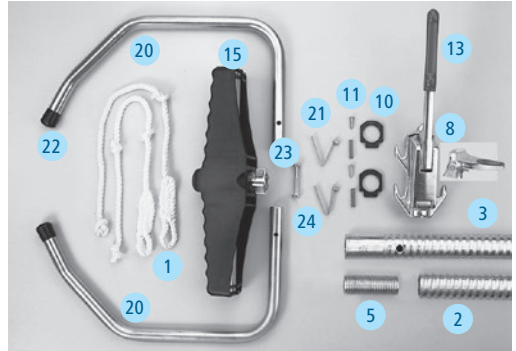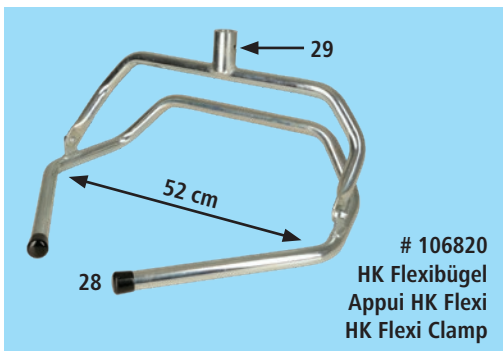
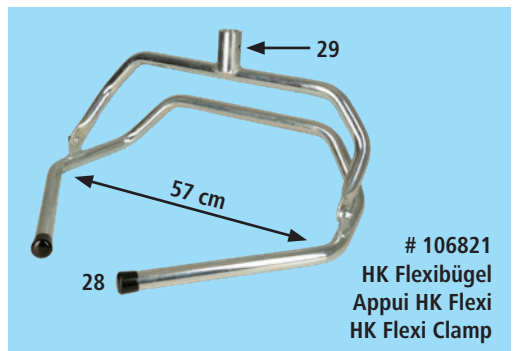
EN List of spare parts

No.	Description	Ref.no. 10121	Ref.no. 10126, 10130, 10131	Ref.no. 10140, 10141, 102060
1	Birth ropes, in pairs	103	103	103
2	Lower tube part with plastic cap	106903	106903	106903
3	Upper tube part with hole	106904	106904	106904
4	Upper tube and lower tube part with intermediate threaded piece and plastic cap	106905	106905	106905
5	Intermediate piece - thread	106907	106907	106907
6	Extension piece, 30 cm in length	106908	106908	106908
7	Plastic cap for the lower tube part	106408	106408	106408
8	Ratchet mechanism 2020 - with plastic cover, no springs and locking plates	106410	106410	-
9	Ratchet mechanism 2060 - reset spring and bolt preassembled, no locking plates	-	-	106811
10	Set of locking plates - without springs and bolts - for ratchet mechanism 2020/2060 threaded tube	106612	106612	106612
11	1 Spring and 1 bolt for ratchet mechanism 2020/2060	106414	106414	106414
13	Plastic cover for the handle of ratchet mechanism 2020	106416	106416	-
14	Set of cover plates and screws for ratchet mechanism 2060	-	-	106817
15	Small plastic support (small head piece) - complete - with grey union nut, bolt and angled pieces	106418	-	-
15	Large plastic support (large head piece) with intermediate tube and aluminium intermediate piece but no supporting-holding bows.	-	106818	106818
16	Grey union nut, screwed, for the small plastic support (small head piece)	106419	-	-
17	Bolt to assemble the small plastic support (small head piece)	106420	-	-
18	Set angled pieces - complete - with screws and nuts	106421	-	-
19	Set screws M 5x16 and nuts for assembling the angled pieces	106422	-	-
20	Supporting-holding bow, in pairs no. 1 and no. 2	-	106823	106823
21	Spring for the supporting-holding bow	-	116	116
22	Plastic cap for the supporting-holding bow	-	106825	106825
23	Intermediate piece, aluminium, for the large plastic support (large head piece) and upper tube part for ref.no. 10126 - 102060	-	106826	106826
24	Screw M 8x55 and nut M8 to assemble the large plastic support (large head piece)	-	106827	106827
25	Locking pin, 6x34 mm in diameter, to assemble the aluminium intermediate piece and the large plastic support (large head piece)	-	106828	106828
26	HK Flexi-clamp for milk breed 52 cm	106820	106820	106820
27	HK Flexi-clamp for meat breed 57 cm	106821	106821	106821
28	Plasticcap for HK Flexi-clamp 106820	106825	106825	106825
28	Plasticcap for HK Flexi-clamp 106821	106829	106829	106829
29	Screw M8 x 50 and nut M8 to assemble the HK Flexi-clamp	106830	106830	106830

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1. Assembly instructions

1.1 General information

Each HK calving aid is partially preassembled. It consists of an upper part with a bore for taking up the head piece and a lower tube part with a cap. A threaded piece is enclosed to screw the bars together. The ratchet mechanism 2020 is always mounted on the upper part of the tube. The mechanism 2060 is either mounted on the upper part of the tube or, when having the model with the HK Flexi clamp, just enclosed. According to the model, the following parts are included: a small head piece with angled corner pieces or a big head piece with a bow, or, in a separate box, the HK-Flexi clamp, either small sized or milk breed or big sized for meat breed. Ropes in pairs are enclosed with each unit.

1.2 Assembly of ref. no. 10121

Screw the upper (3) and lower (2) part of the tube together using the threaded piece (5). Insert the grey union nut (16) with the small opening first on the upper part of the tube. Attach the head piece (15) and twist until the hole lies directly above the hole in the upper part of the tube. Place the bolt (17) into the hole and screw the grey union nut (16) tightly on to the head piece. Mount the angled pieces (18) with screws and nuts (19) in each case on the left and the right of the head piece.

1.3 Assembly of ref.no. 10126 and 102060

Screw the upper (3) and the lower (2) part of the tube together using the threaded piece (5). Attach the head piece (15) to the upper (3) part of the tube and twist until the hole of the aluminium link (23) lies directly above the hole of the upper part (3) of the tube. Push the screw M8 x 55 (24) through the aluminium link (23) and the upper part of the tube (3) and fix with nut. Stick the bows (20) into the lateral opening of the head piece. Push in the spring head, telescope the bows to the desired dimension and let it lock in the bores provided.

1.4 Assembly of 10126 and 102060 with HK Flexi clamp

When having the ratchet mechanism 2060 (9) with the HK Flexi clamp, the ratchet has to be mounted on the upper part of the tube. Therefore, the release lever (see operating instructions Fig. 5 b) must be pressed and held in this position. After that, attach the HK Flexi clamp to the upper part of the tube (3), twist until the hole lies directly above the hole of the tube and fix with screw M8 x 50 (29) and nut.

1.5 After assembly

Pull the ratchet mechanism over the upper and the lower part of the tube by moving the handle up and down. Reset the ratchet mechanism to its original position by releasing the locking device (see operating instructions).

2. Operation

2.1 Fundamental points regarding the calving aid

Applying mechanical pulling aid requires a sound knowledge of the process of birth. Be patient and do not interfere too early. If the birth takes a normal course assistance is necessary only 1 to 2 hours after bursting of the amniotic sac (waterbag). Mechanical pressure is reasonable only if, for head presentation, the two front limbs and the head of the calf are stretched out and if, for breech presentation, the rear limbs and the pelvis are stretched out and have enough space in the pelvis of the mother. For other presentations and presentations that cannot be corrected as well as in case of an oversized foetus a vet should be consulted. In each situation you should apply pressure carefully, that means pull in small doses in accordance with the contractions and pay attention to pull in the correct direction (see pos. 6). Otherwise you run a severe risk of hurting the mother or the calf.

2.2 After installation of your HK Calving Aid according to the instructions proceed as follows:

- 2.2.1 Arrange a clean place to lie the new born calf preferable with straw. Also take care that you have enough space for fixing your HK Calving Aid. Clean the bearing of the animal with a neutral soap and rinse with clean water.
- 2.2.2 Thoroughly examine the cow to get an exact overview of the position of the calf. Pay attention to work with utmost cleanliness. Wash your hands and arms and apply a sterile lubricant before carrying out any correcting examination.
- 2.2.3 Thoroughly clean the calving ropes before each application. Make a loop, place it around the pasterns (Fig. 1) and tighten it.
- 2.2.4 Arrange the attachment support of the HK Calving Aid crosswise directly under the orifice of the vagina (Fig. 2). If you use the support yoke (ref.no. 10126, 102060) adjust it according to the pelvis width to make sure it does not slip to the bottom. For adjustment of the support yoke press the spring studs respectively, push to the required width and let them engage in the holes provided. When having the HK Flexi clamp, either small sized for milk breed (52 cm) or big sized for meat breed (57 cm), push it according to the pelvis in a way that the movable part of the clamp lies directly under the orifice of the vagina. Caused by that all kinds of slipping will be avoided.
- 2.2.5 Suspend the calving ropes at the hooks of the clamping mechanism via the prepared knots (Fig. 3). It should be as short as possible. Choose the hooks that are arranged at the same height opposite each other if you want to pull both limbs at the same time. Alternatively if you prefer to pull one limb at a time attached to the hooks that are arranged opposite each other. The ratchet mechanism 2060 has only two hooks for pulling simultaneously at both limbs. **Please note:** According to the latest scientific knowledge pulling should be effected simultaneously at both limbs. Careful working makes the passage of the shoulders and the pelvis easier. Moreover, application of the entire pulling strength on one hoof could possibly cause injuries and should be avoided

2.2.6 While observing the contractions move the handle of the ratchet mechanism up and down and assist by pulling in a controlled manner. If possible, take care that the ropes / chains are always stretched. The correct pulling direction is very important to achieve the desired success of the mechanical pulling aid. In the beginning pull straight in accordance with the extension of the spinal column of the mother (see direction of the arrow A on Fig. 4). When the head and the shoulders (or the pelvis in case of breech presentation) have come out pull towards the udder by slowly tilting the pulling bar (see direction of the arrow B on Fig. 4) downwards.

2.3 Quick release system of mechanism 2020/2060.

You have made up your mind for the advantages of the quick release system the handling of which is very easy. Press down the release lever (Fig. 5 b) with your thumb. To reach the starting position of the ratchet mechanism the release lever must be pressed down. With your second hand simultaneously pull the handle downwards to the reach locking plate and hold it in this position. The ratchet can now easily be pushed in the direction of the attachment support. Please note: In case of an extreme pulling load we recommend to push the handle of the ratchet upwards in the direction of the attachment support first (see direction of the arrow 'a' on Fig. 5). Thus the rear chamber is relieved. After that push the release lever (see Fig. 5 'b') and push the handle downwards to reach the locking plate (see direction of the arrow 'c' in Fig. 5). Hold it in this position and move the mechanism to the starting position.

2.4 Cleaning and care

Thoroughly clean the equipment after each application. Grease the pressure springs and bolts now and again. To make possible unscrewing of the Calving Aid for transportation without problems you should also grease the intermediate piece - thread (see item 5 of the spare parts list) some time or other. According to their use the top and bottom parts of the rods and locking plates (see item 2, 3 and 10 of the spare parts list) are subject to natural wear and tear.

2.5 Important instructions for model 2060: Setting the traction power

2.5.1 The principle of setting the traction power: You have decided to buy the model 2060 and thus opted in favour of the gentle calving aid.

The mechanism of this calving aid has an adjustable traction power control system and allows pulling forces to be applied in two stages, from 50 to 100 daN and from 100 to 160 daN. When the preset traction power has been reached, the handle is locked. This reduces the risk of injuries that can result from unchecked pulling. You should only override this limit in exceptional cases. The traction power measured according to the DLG test report No. 2975 then reaches maximum values of more than 500 daN.

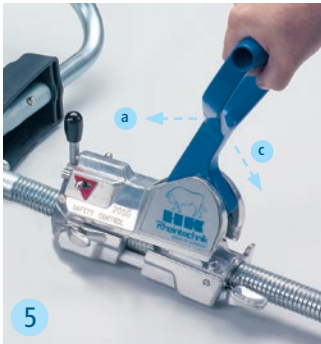
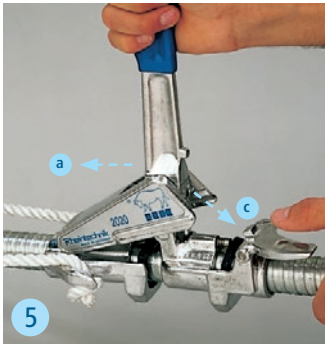
2.5.2 Setting the traction power: To set the traction power, the limiting lever (lever with the black top) should be placed in the desired position. Initial settings for the pulling aid should not exceed 100 daN. Once this traction power has been reached, the handle is locked. You should then wait for the animal's next contraction. If the handle continues to be locked despite the next contraction having started, the traction power is too low. The next-higher setting should be chosen, using the limiting lever. To do this, use your other hand to push down the pin next to the number 100. The setting up to 160 daN is sufficient to successfully complete the great majority of births. It may be necessary, in a few individual cases, to override the traction power limit. To do so, press down the pin in the red triangle and at the same time move the limiting lever. **Warning:** When using the maximum possible traction power, there is a high risk of injury to the mother and calf. We suggest that you consult a veterinary surgeon before overriding the traction power limit.

3. Warranty

Based on our General Terms and Conditions of Business, we offer a 2 years warranty for this device covering damage caused by material or manufacturing defects. This warranty begins on the date of purchase. The warranty does not cover:

1. Damaged caused by natural wear and tear (wear parts, sliding parts), excessive strain and incorrect use (non-compliance with operating instructions).
2. Damage caused by force majeure or sustained during transport.
3. Damage to rubber parts, e.g. on connecting lines. Warranty repairs are only carried out by the manufacturing plant or an authorised contracted workshop.

The manufacturer reserves the right to reject any warranty claims in the event of non-compliance. The device must for this purpose be returned to the point of sale, at the risk of the device owner, complete, packaged and with sufficient postage, including the original invoice (address see dealer stamp). Please provide as much information as possible on the nature of the problem. Costs incurred as a result of unjustified defect claims will be invoiced to the device's owner.

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(EN)

- 1 Fixing the ropes
- 2 Fixing the attachment support
- 3 Hanging on the ropes
- 4 Pulling during the process of birth
- 5 Quick release mechanism 2020/2060